

DATE

25-March-2020

TO

Barco Healthcare Display System Customers

FROM

Albert Xthona & Inge Haesaerts, Product Managers

Addendum to Barco Display Cleaning Instructions

Dear Customers,

Many people would like to use stronger concentrations of alcohol-based products to clean their Coronis and Nio display systems. In response, we have tested higher concentrations of alcohol and you should now feel free to use up to 70% solutions of ethanol or isopropyl alcohol as shown below in the annotated section from the User Guide.

Cleaning your display


5.1 Cleaning instructions


To clean the display

Clean the display using a sponge, cleaning cloth or soft tissue, lightly moistened with a recognized cleaning product for medical equipment. Read and follow all label instructions on the cleaning product. In case of doubt about a certain cleaning product, use plain water.

Do not use following products:

- Alcohol/solvents at higher concentration > ~~5%~~ 70%
- Strong alkalis lye, strong solvents
- Acid
- Detergents with fluoride
- Detergents with ammonia
- Detergents with abrasives
- Steel wool
- Sponge with abrasives
- Steel blades
- Cloth with steel thread

 **CAUTION:** Take care not to damage or scratch the front glass or LCD. Be careful with rings or other jewelry and do not apply excessive pressure on the front glass or LCD.

 **CAUTION:** Do not apply or spray liquid directly to the display as excess liquid may cause damage to internal electronics. Instead, apply the liquid to a cleaning cloth.

This is applicable to models with {MDCC, MDCG, MDMC, MDMG, MDNC, MDNG, MDRC}, e.g. MDMC-12133, MDCC-6530, manufactured after 1-January 2008.

Regards,
Albert & Inge